## Description

EN

- ① Cuff
- ② Cuff Tube
- ③ Cuff Connector A
- ④ Cuff Connector B

## Choosing the connector

- For blood pressure monitors that have a white, silver or dark coloured cuff socket on the device, use **Connector A** ③ with the grey connector pin.
- For blood pressure monitors that have a yellow coloured cuff socket on the device, use **Connector B** ④ with the yellow connector pin.
- For the following models, use the **original cuff connector**, that has been supplied with the blood pressure monitor: Microlife VSA (BP3GP1-1L)
- **No connector:** Semi-automatic blood pressure monitors which have a tube shaft directly on the device. Simply plug the tube to that shaft without a connector.

☞ To change the connector, pull the connector out of the cuff tube and insert the required connector into the cuff tube as far as it will go.

## Cleaning the cuff

1. Remove the cuff connector ③ from the cuff tube ② and carefully pull the bladder through the opening at the edge of the cuff cover.
2. Hand wash the cuff cover in soapsuds: not hotter than 30 °C.
3. Completely dry the cuff cover by linen drying.
4. Loop the cuff tube back through its opening and carefully place the bladder flat in the cuff cover.
5. Reattach the cuff connector on the cuff tube.

☞ The bladder must lay straight in the cuff cover, not folded.

☞ Do not use fabric softener.

⚠ **WARNING:** Do not wash the cuff in a washing machine or dishwasher!

⚠ **WARNING:** Do not dry the cuff cover in a tumble dryer!

⚠ **WARNING:** Under no circumstances may you wash the inner bladder!

⚠ The cuffs are sensitive and must be handled with care.

## Garantie

- The cuff has a functional guarantee (bladder tightness) for 2 years.
- Not included are sleeve and cuff enclosure due to wear.
  - The guarantee is only valid on presentation of the purchase receipt.
  - The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.
- The cuff has an average service life of 2 years or 5000 measurements, whichever comes first.

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